

Customize Battery in CANTAB

1. Open the Control Centre.
2. Click on “Customize Battery.”
3. In the “CANTAB Battery Editor” box that pops up, choose “New Battery” and click on “OK.”
4. Change the “Battery Name:” to “Consortium Battery.”
5. Highlight the name of the test(s) you are interested in, and click “Add->.”
6. The tests should be in the following order:
 - a. Motor Screening
 - b. Big/Little Circle
 - c. Pattern Recognition Memory
 - d. Spatial Recognition Memory
 - e. Spatial Span (if administering)
 - f. Spatial Working Memory
7. For each test, when highlighted, make sure in the box marked “Setup:” you choose “Clinical (Standard).” For PRM and SRM, make sure in the “Stimulus” box it is set to “0.”
8. Click on “OK.”

Congratulations! You have just created a new battery for the CIFASD tests.